

Precaution on Osseointegrated Prostheses on ALLUX™

Assemble the ALLUX™ and the other prosthetic component according to the following procedure.

[Assembly Procedures]


- Ensure that the prosthetic foot is in contact with his/her buttock at the flexion angle of ALLUX™ . (155degree)
- Ensure material such as rubber or cushion form must be attached on contact area between the adapter/pipe and knee joint to avoid any direct contact.

NOTICE

If the above mentioned requirement is not available for the structure of prostheses, inform the following instructions to the user as noticed point.

- 1) Do not apply a strong impact on the knee joint at flexing the knee at the maximum flexion angle.**
- 2) Do not apply loading over the maximum weight limit for ALLUX™ at the maximum flexion angle.**

The buttock and prosthetic foot must be contacted at the maximum flexion angle.

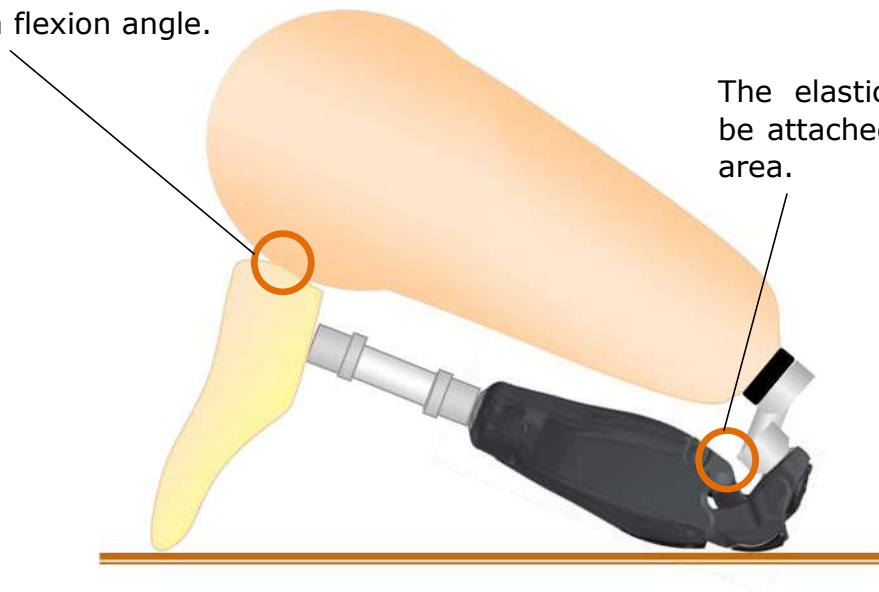

The elastic material must be attached on the contact area.