# Slide Connector N-D114

## Instruction Manual

### 1. Information

This product must be assembled by a certified prosthetist. Please read this document carefully before using the product and observe the safety notices.

Instruct the user in the safe use of the product.

Please contact the manufacturer if you have questions about the product or in case of problems.

Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.

Please keep this document for your records.

### 2. Purpose of use and product overview

This product is a connector designed to adjust anterior-posterior alignment above the knee joint on trans-femoral prostheses and to adjust anterior-posterior and medial-lateral alignment on trans-tibial prostheses. It is for everyday life and can be used multiple times by one user.

The connection with other prosthetic limb parts is a combination with a female pyramid on the upper side and a male pyramid on the lower side.

You can adjust the angle of the sagittal plane and frontal plane of the prosthesis foot and slide back and forth or left and right.

It does not support medial-lateral direction slide adjustment of trans-femoral prostheses.

### 3. Product specifications

- Maximum body weight: 100 kg (ISO10328-P5-100kg)
- K Level (MOB): K1 Level (MOB1) ~ K3 Level (MOB3)
- Expected lifetime: 3 years
- Applicable environmental conditions
  - Storage temperature range: -20°C to +60°C (-4°F to +140°F)
  - Operating temperature range: -10°C to +40 °C
  - Chemicals/liquids: Water or liquid (salt water ,sweat, urine) splash
- Prohibited environmental conditions
  - Usage environment: Underwater (in liquid)
  - Chemicals/liquids: Hydrochloric acid, sulfuric acid

### 4. Safety

Explanation of warning symbols

|  |  |
|--|--|
|  | Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.  |
|  | Indicates a potentially hazardous situation which, if not avoided, could result in minor or moderate injury. |
|  | Indicates actions that you must not do when handling the product.  |

Strictly observe the following instructions to use the product safely.

**WARNING**

- When using the connector on a transfemoral prosthesis, make sure the below leg touches the socket or buttocks in maximum flexion.
- \* If it does not touch in maximum flexion, an excessive load will be placed on the parts, which could result in breakage.
- Tighten each of the set screws with the designated torque.
- \* If the torque used to tighten the screws is less than the designated amount, the set screws could become loose and fall out during use.
- Adjust the alignment according to the instructions of the connected device.

**PROHIBITION**

- DO NOT use for a person who weighs over 100 kg (220 lb).
- DO NOT attach the sliding connector for medial-lateral direction on trans-femoral prostheses.
- DO NOT remove the slide stopper bolt to adjust the sliding motion to more than 22 mm.
- DO NOT use 2 pieces or more of the slide connectors together on a prosthesis.
- \* Failure to observe the prohibitions could result in damage to the products and user injury.

**CAUTION**

- Do not leave or use the product in an environment not applicable.
- If there is any abnormality such as damage, abnormal noise, or backlash, discontinue use and take appropriate measures. (Contact the manufacturer or specialist workshop)
- Do not use the product beyond its useful lifetime.

### 5. How to use

The amount that the N-D114 Slide Connector can slide is 22mm at the maximum. The position of the sliding part can be fixed at any position within the 22 mm of its maximum.

After adjusting the angle of alignment, tighten the pyramid set screw (2).

After adjusting the amount of sliding, tighten the slide set screw (1).

\* Apply the adhesive (equivalent to LOCTITE 243) on the thread part for anti-loosening of each screw, and tighten by a specified torque.

**CAUTION**

- DO NOT touch the rail when assembling the prosthesis or adjusting the slide. Doing so could cause your fingers to get pinched and result in injury.

### 5.1. How to fix the slide fixing screw and the female pyramid fixing screw


(1) Slide set screw (M8 x 8): 2 pieces

Tightening torque: 10 N·m

(2) Pyramid set screw (M8 x 12): 4 pieces

Tightening torque: 10 N·m

\* Slide limitation bolt (on inside) must not be removed.

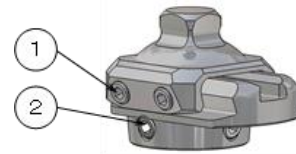### 5.2. Cleaning

If necessary, wipe the surface with a soft cloth moistened with fresh water.

### 5.3. Maintenance

For safety, carry out regular inspections once a year for loose screws or abnormalities.

### 5.4. Disposal

This product is made of titanium alloy and stainless alloy. Observe the information provided by the responsible authorities in your country regarding disposal procedures.

### 6. Warranty

- The warranty for this slide connector is valid for a period of one year from the date of purchase.
- Note that if the connector is broken or damaged due to improper use the warranty may become void even if it happens during the warranty period.
- Please note that we cannot guarantee if the product is malfunction or damaged due to use outside the range shown in this manual.

### 7. Technical data

- Weight: 195 g
- Overall height: 43.8 mm
- Slide amount: 22 mm

This device complies with the European Medical Device Regulation (EU) 2017/745.

### Manufacturer

Nabtesco Corporation  
Accessibility Innovations Company  
Assistive Products Department  
35, Uozakihama-machi, Higashinada-ku Kobe 658-0024, Japan  
Phone: +81-78-413-2724 Fax: +81-78-413-2725  
URL: <https://welfare.nabtesco.com>

### Authorized representative for EU countries

PROTEOR SAS  
6 rue de la Redoute  
21850 Saint-Apollinaire, France  
Phone: +33 3 80 78 42 42 Fax: +33 3 80 78 42 15  
[cs@proteor.com](mailto:cs@proteor.com)

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